


# Connecting a Luer-Lock PD Bag to the APD Luer-Lock Adapter

## Important things to remember


**1** Place the non-Fresenius PD solution with Luer-Lock bag connector on a clean surface with connector aimed towards you.

**2** Remove APD Luer-Lock adapter from package.


**3** Pull off the pull ring (E) from the non-Fresenius PD solution bag (F).

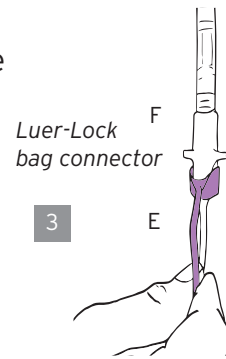


**4** Unscrew the blue cap (C) from the APD Luer-Lock adapter (B) and discard the blue cap.

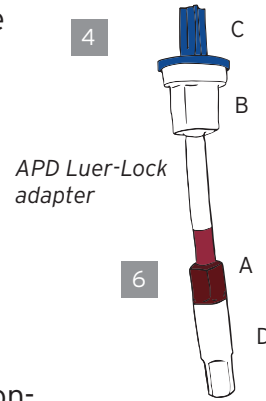


**5** Connect the APD Luer-Lock adapter to the non-Fresenius PD solution bag by turning the APD Luer-Lock adapter (B) clockwise onto the non-Fresenius PD solution bag (F) connector until tight. Note that (F) is prone to touch contamination and the grip should remain as high as possible.

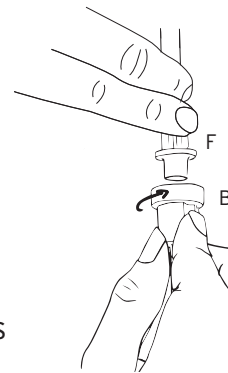

**6** Remove the white cap (D) from the APD Luer-Lock adapter (A). Ensure that the blue cap is removed from the cyclor line with the green clamp (H).

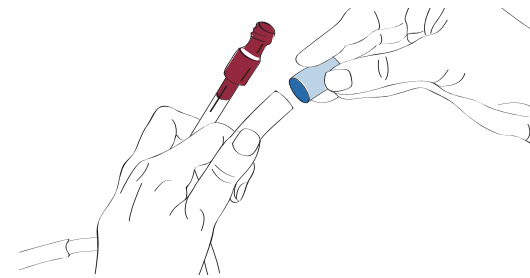

**7** Hold the APD Luer-Lock adapter (A) steady. Connect the cyclor line with green clamp (H) by turning clockwise. Note that (A) is prone to touch contamination and grip should remain as high as possible.

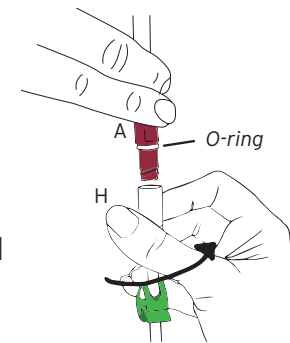

**8** Make sure that the O-ring on the red connector of the APD Luer-Lock adapter is covered. If the O-ring is exposed, partially unscrew the connector and re-attach securely.

**9** Check that the connection between APD Luer-Lock adapter (A) and the last bag line (green clamp) of the cyclor set (H) is tight.

**10** When prompted by the cyclor, break the cone of the PD solution bag. Ensure that the connection between the non-Fresenius PD solution bag and APD Luer-Lock adapter is tight.

**11** Dispose used product as per local regulations.

**Indications for Use:** The Fresenius Medical Care APD Luer-Lock adapter is intended for use with an APD cyclor for the exchange of peritoneal dialysis solution. The single-use device is designed to connect any Fresenius Medical Care cyclor set with a Safe-Lock® connector to a peritoneal dialysis solution bag equipped with a standard Luer-Lock connector. The Fresenius Medical Care Liberty® cyclor is intended for use in acute and chronic peritoneal dialysis.

**Caution:** Federal (US) law restricts this device to sale by or on the order of a physician or other licensed practitioner. Read the Instructions for Use for safe and proper use of this device. For a complete description of hazards, contradictions, side effects and precautions, see full package labeling at [www.fmcna.com](http://www.fmcna.com).

**FMCNA.COM**

Fresenius Renal Technologies, a division of Fresenius Medical Care North America | 920 Winter Street, Waltham MA 02451 | Customer Service: 800-662-1237 | Technical Support: 800-227-2572

©2013-2019 Fresenius Medical Care North America. All rights reserved. Fresenius Medical Care, the triangle logo, Liberty, and Safe-Lock are trademarks of Fresenius Medical Care Holdings, Inc. and/or its affiliated companies. All other trademarks are the property of their respective owners. P/N 102039-01 Rev B 10/2019

