

# Quick Start: Liberty® Select Cycler

## First-time use

Ensure cycler is plugged into wall outlet, stay•safe® organizer is attached to the mounting tab below the screen, optional USB drive (also known as IQdrive™) is inserted, and optional modem or Kinexus™ gateway is connected.

✓	STEPS
	<b>Gather supplies</b>
	<b>Set up for treatment</b>
	Turn on cycler
	Check treatment settings
	Press OK to start
	Mask, then wash hands
	Open cycler set package and use caution to not touch the cassette film. Inspect for damage.
	Open cassette door and insert cassette
	Insert stay•safe PIN connector (Patient Line with blue clamp) into stay•safe organizer
	Choose heater bag volume and press NEXT
	Remove overwrap from solution bag
	Check solution bags for S.C.A.L.E.
	Place solution bags on a flat surface (on top of cycler)
	Clean hands
	Connect line with red clamp to heater bag
	Connect lines with white clamps to additional solution bags if needed
	Connect line with green clamp to solution bag for Last Bag Option, if option is enabled
	Place heater bag on the heater tray and move other connected solution bags to a flat surface or the Liberty cart
	Connect line with yellow clamp to drain line or drain bags. Press Next.



✓	STEPS
	If using drain line, remove cap and place end in appropriate area to drain
	Break cones on all bags
	Clamp unused lines
	Press NEXT
	Verify connected bags
	Press NEXT
	After flush is complete, ensure blue clamp on patient line is open
	Press NEXT
	Check patient line to make sure it is primed
	Press NEXT
	Press BACK to prime again (if needed)
	Close the blue clamp on patient line
	Enter daytime manual exchange volume, if you did one, then press NEXT. Confirm it, then press NEXT.
	<b>Connecting yourself for treatment (use aseptic technique)</b>
	Place catheter extension set into the “Extension” notch of stay•safe organizer
	Clean or wash hands
	Remove the clear cap from the stay•safe PIN connector
	Remove catheter extension set from cap
	Connect catheter extension set to PIN connector
	Place clear cap removed from PIN connector onto used cap and discard
	Open blue clamp on patient line
	Open white clamp on catheter extension set
	Press NEXT to begin treatment
	<b>Ending your treatment (use aseptic technique)</b>
	Mask, then clean or wash hands
	Place new stay•safe cap into “New Cap” notch
	Turn blue button on PIN connector ¼ turn clockwise, then press it in. Check that the PIN is fully engaged.
	Close blue clamp on patient line, then white clamp on catheter extension set
	Place PIN connector in organizer
	Place catheter extension set into the Extension notch of the stay•safe organizer
	Remove protective cover from stay•safe cap
	Unscrew catheter extension set from PIN connector and ensure PIN is in place
	Immediately connect catheter extension set to stay•safe cap
	Remove catheter extension set from organizer



✓	STEPS
	<b>Ending your treatment (use aseptic technique) cont.</b>
	Secure catheter extension set to abdomen
	Press NEXT
	<b>Documenting your treatment</b>
	Enter patient data, if applicable
	Press OK to save after each entry
	Review treatment summary and press NEXT
	Close all clamps on cyclor set and drain line/drain bags
	Press NEXT
	Open cassette door, remove cassette, then gently close cassette door
	Dispose of empty solution bags, unused solution, and waste as instructed by your home therapy nurse
	Press NEXT
	Turn off the cyclor
	<b>Keeping your cyclor clean</b>
	Wipe cyclor with a soft cloth moistened with a 1 to 100 solution of bleach water or other hospital disinfectant. Rinse off cleaning solutions with a water-dampened cloth.

Read the Instructions for Use for safe and proper use of this device. For a complete description of hazards, contraindications, side effects, and precautions, see full package labeling at [fmcna.com](http://fmcna.com).

## Indications for Use

The Liberty Select cyclor is indicated for acute and chronic peritoneal dialysis.

## Caution

Federal (U.S.) law restricts this device to sale by or on the order of a physician.

