



Liberty[®] Select Nurse In-Service Checklist

For software versions 3.0.2 or higher

Instructions

Use this form to track and document patient and care partner training on Liberty Select cyclor software Version 3.0.2 and higher. This checklist is designed to be used as a resource in conjunction with the Liberty Select cyclor User's Guide. Use a check mark to show steps performed during review and practice. Initials indicate all steps have been completed for each skill in that section. If the skill is not adequately demonstrated for a sub-topic, a follow-up plan of action is recommended. Do not leave fields blank. Enter N/A (not applicable) where appropriate.

CENTER NAME			
TRAINING START DATE		TRAINING END DATE	

Signatures

LEARNER NAME

PRINTED NAME			<input type="checkbox"/> In Person <input type="checkbox"/> Virtual
DATE		SIGNATURE	

TRAINER NAME

PRINTED NAME			<input type="checkbox"/> In Person <input type="checkbox"/> Virtual
DATE		SIGNATURE	

► Module 1: The Cyclor's Components

Skills	Reviewed		Learner Demonstrates Skill	
Rear View:				
Power switch	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Power cord/power cord socket	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Fan vent	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
If applicable: RS232 port for Kinexus Gateway or modem	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Side View:				
Handles	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
USB drive (Also known as IQdrive™)	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Front View:				
Heater tray	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Heater bag sensor	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Technical Support number	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Cyclor serial number	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Software version number	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► Module 1: The Cyclor's Components con't

Skills	Reviewed	Learner Demonstrates Skill
Touchscreen	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Control Panel: • UP/DOWN Arrow keys • OK button • STOP key	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Cassette door	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Pump heads	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Door sensor	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Safety bar	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Pressure sensors	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Cassette guide pins and cassette catch	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Cassette pump plate, valves	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Door latch	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Status bar: blue, green, yellow, orange, red	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Touchscreen functions: screen-buttons, movement-buttons, parameter-buttons	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► Module 1: The Cyclor's Components con't

Skills	Reviewed	Learner Demonstrates Skill
Disposables:		
stay•safe® Organizer with clip	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
stay•safe catheter extension set	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Liberty cyclor set	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Cassette/cartridge	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Drain line (yellow clamp)	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Heater bag line (red clamp)	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Solution bags (white clamps)	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Last Bag (green clamp)	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Patient line (blue clamp)	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• stay•safe PIN connector(s)	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Liberty drain set	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Multiple Tubing Set (MTS)	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► Module 2: Navigating with the Screen Buttons

Skills	Reviewed	Learner Demonstrates Skill
My Records button	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
My Treatment button	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
My Settings button	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

► Module 3: Setting Up the Cyclor for Treatment

Skills	Reviewed	Learner Demonstrates Skill
Turn power switch to ON .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Touch My Settings button.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Select program mode:		
• Cycle-based	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Treatment-based	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Select therapy type and enter values (if not using IQdrive/USB drive):		
• CCPD: Treatment-based	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• CCPD: Cycle-based	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► **Module 3: Setting Up the Cyclor for Treatment con't**

Skills	Reviewed	Learner Demonstrates Skill
• IPD: Treatment-based	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• IPD: Cycle-based	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• PD+: Treatment-based	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• PD+: Cycle-based	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Tidal: Treatment-based	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Tidal: Cycle-based	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

► **Module 4: Entering Patient Settings**

Skills	Reviewed	Learner Demonstrates Skill
Name	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Date of birth	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Weight	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Gender	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► Module 5: Entering Options Settings

Skills	Reviewed	Learner Demonstrates Skill
Select values		
Add diurnal UF	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Language (English, Spanish, or French)	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Last bag option	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
SI units (yes or no)	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Personal data	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
USB-enabled	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Drain exit %	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Add drain	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Add drain alert	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Flow Alert	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

▶ Module 6: Entering Hardware Settings

Skills	Reviewed	Learner Demonstrates Skill
Select values		
Alarm loudness	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Key loudness	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Date format	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Date	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Time format	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Time	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Screen blanking	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Brightness	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

▶ Module 7: Initiating Treatment

Skills	Reviewed	Learner Demonstrates Skill
Check that cyclor and stay•safe® Organizer and blue clip have been cleaned per manufacturer’s recommendations.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Ensure cyclor is plugged into a grounded, three-pronged electrical outlet and that power is on.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Verify treatment values on “Ready” screen.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Press OK key to start treatment.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Gather supplies, mask, and wash hands.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Inspect cyclor set for any signs of damage to tubing, clamps, PIN connector(s), or film before inserting it into the cyclor. Check that the cassette and tubing are free from kinks or damage. Look for scratches, loose tubing, or holes in the cassette film, and check for any damage to clamps or PIN connector(s). Note: If cyclor set is damaged, do not insert into the cyclor. Save the cyclor set and call Technical Support.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Ensure cyclor set is properly inserted: The cassette is under each of the upper metal guide pins and snapped into the lower catch. Ensure lines hang straight.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
<ul style="list-style-type: none"> First, carefully insert the top portion of the cassette. Then, press the lower portion in until it snaps securely, and close the door. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

▶ Module 7: Initiating Treatment con't

Skills	Reviewed	Learner Demonstrates Skill
<ul style="list-style-type: none"> Insert the PIN connector from the patient line (blue clamp) into the stay•safe Organizer. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Inspect all solution bags, including heater bag, for S.C.A.L.E.* ; connect solution bags to lines and properly place heater bag on heater tray (* S olution, C larity, A mount, L eaks, E xpiration date).	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
<ul style="list-style-type: none"> Select heater bag volume you are using for treatment. (Use the largest solution bag and must be 1000mL more than fill volume). 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Connect drain lines or drain bags; touch Next .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Break cones (use stay•safe Handi-tool as necessary). No need to float the cone.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Clamp unused solution lines; touch Next .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Verify bag connections; touch Next .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Cycler performs a tubing flush.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Ensure patient line (blue clamp) is open; touch Next to prime.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► Module 7: Initiating Treatment con't

Skills	Reviewed	Learner Demonstrates Skill
<ul style="list-style-type: none"> When using a 20-foot-long patient line, priming needs to be repeated by pressing the Back button again to ensure the patient line is fully primed. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
When priming is complete, close the blue clamp and press Next . (Press Back on screen, if needed, to repeat priming.)	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Select daytime manual exchange information.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
<ul style="list-style-type: none"> If the patient has not performed a manual exchange fill since they last turned off the cyclor, set the daytime exch. button to NO. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
<ul style="list-style-type: none"> If the patient has performed a manual exchange fill since they last turned off the cyclor, set the daytime exch. button to YES. Add manual fill volume. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
<ul style="list-style-type: none"> If the patient drained last fill from the cyclor or performed a daytime drain, set the daytime exchange button to YES with manual fill Volume set to 0mL. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Confirm the daytime manual exchange information.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Use aseptic technique (should still be masked and re-wash hands per center's protocol) to connect the patient's catheter extension set to the patient line (blue clamp) of the cyclor set.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► Module 7: Initiating Treatment con't

Skills	Reviewed	Learner Demonstrates Skill
Open blue clamp on patient line and white clamp on catheter extension set; touch Next .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Bypass a phase of the cycle.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
"Bypass not allowed at this time" message.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Pause: Disconnect from cyclor (if prescribed); touch Next .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Pause: Reconnect to cyclor (re-mask and re-wash hands per facility protocol); touch Next .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

► Module 8: Ending Treatment

Skills	Reviewed	Learner Demonstrates Skill
Aseptically disconnect per center's protocol with PIN technology:		
• Ensure patient line and catheter extension set are open.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Turn the blue button on the PIN connector ¼ clockwise to unlock.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Push in the blue button to insert the PIN into the catheter extension set.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Close blue clamp on patient line and white clamp on catheter extension set.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► Module 8: Ending Treatment con't

Skills	Reviewed	Learner Demonstrates Skill
<ul style="list-style-type: none"> • Insert PIN connector attached to the catheter extension set into the stay•safe clip of the Organizer. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
<ul style="list-style-type: none"> • Remove protective cover from new stay•safe cap. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
<ul style="list-style-type: none"> • Unscrew catheter extension set from PIN connector. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
<ul style="list-style-type: none"> • Pull catheter extension set back from the PIN connector. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
<ul style="list-style-type: none"> • Hold the catheter extension set up slightly—be sure the PIN is visible. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
<ul style="list-style-type: none"> • Connect catheter extension set to the new stay•safe cap. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
<ul style="list-style-type: none"> • Remove catheter extension set from the Organizer. Make sure cap is tight. 	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Touch Next to continue.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Update personal data.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Close all clamps and touch Next .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► Module 8: Ending Treatment con't

Skills	Reviewed	Learner Demonstrates Skill
Remove cassette and touch Next .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Demonstrate safe disposal of effluent and tubing.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Review treatment summary, then touch Next .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Details	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• My Records	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Describe B, P, S, and F in My Records	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Power down cyclor	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

► Module 9: Using the IQdrive (USB Drive)

Skills	Reviewed	Learner Demonstrates Skill
USB-enabled option set to YES .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
First insert IQdrive/USB drive, then turn on cyclor.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► Module 9: Using the IQdrive (USB Drive) con't

Skills	Reviewed	Learner Demonstrates Skill
USB upload – Accept – Reject – Current Settings.	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
Kinexus Gateway or wireless modem (optional).	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
Using IQdrive (USB drive) with Optional Kinexus Gateway.	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
• USB-enabled option set to YES .	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
• First connect the Kinexus Gateway into a wall outlet, (wait until the LED turns green) then continue to connect to the cyclor.	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
• Insert USB drive, then turn on cyclor.	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
• Use new Kinexus Gateway settings? – Accept – Reject – Current Settings.	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>

► Module 10: Troubleshooting Common Warnings and Alarms

Skills	Reviewed	Learner Demonstrates Skill
Press STOP to mute audible signal/alarm.	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
Error messages (example Drain complications) Review Meaning and Action Required.	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► Module 10: Troubleshooting Common Warnings and Alarms con't

Skills	Reviewed	Learner Demonstrates Skill
Warnings – press STOP to mute; follow instructions on the screen.	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
Alarms – press STOP to mute; record alarm code.	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>

► Module 11: STAT Drain (Unplanned Drain)

Skills	Reviewed	Learner Demonstrates Skill
Where to find STAT Drain box.	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
Confirm STAT Drain .	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
When can a STAT Drain be performed during treatment?	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
Screen display during STAT Drain .	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
What happens after a STAT Drain .	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
Select STAT Drain option.	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>
Confirm STAT Drain .	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► Module 11: STAT Drain (Unplanned Drain) con't

Skills	Reviewed	Learner Demonstrates Skill
Screen display during STAT Drain .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
A STAT Drain works the same as a regular drain. You may stop it or bypass it as needed.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

► Module 12: Power Failure

Skills	Reviewed	Learner Demonstrates Skill
Power failure notification.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Power restored.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Safety system check.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Power fail recover successful.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
i. Resume treatment.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
• Recovery failure.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
i. Cancel treatment.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
ii. Disconnect with PIN technology (see page 3, "Ending Treatment").	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► Module 12: Power Failure con't

Skills	Reviewed	Learner Demonstrates Skill
iii. When prompted, open door and remove cassette.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
iv. Restart entire treatment with fresh dialysate bags and new Liberty cyclor set.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

► Module 13: Cleaning the Cyclor

Skills	Reviewed	Learner Demonstrates Skill
Clean the exterior surface using a damp cloth and a dilute (1:100) bleach solution or other suitable hospital disinfectant.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Rinse off cleaning solutions with a water-dampened cloth.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Periodically check for damage to cabinet and loose or missing pieces.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

► Module 14: Liberty Select Support Services

Skills	Reviewed	Learner Demonstrates Skill
Technical Support: 800-227-2572	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Clinical Resource Line: 877-954 -3340	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS

LEARNER INITIALS

DATE

► **Module 15: Adapting PD Catheter Connections to Alternate PD Delivery Systems**

Skills	Reviewed	Learner Demonstrates Skill
Adapting PD Catheter Connections to Alternate PD Delivery Systems P/N 101419-01	Yes No <input type="checkbox"/> <input type="checkbox"/>	Yes No <input type="checkbox"/> <input type="checkbox"/>

Caution: Federal (U.S.) law restricts these devices to sale by or on the order of a physician.

Intended Use: The Kinexus Gateway is intended for transmission of patient data collected during peritoneal dialysis therapy with the Liberty Select cyclor.

Note: Read the Instructions for Use for safe and proper use of this device. For a complete description of hazards, contraindications, side effects, and precautions, see full package labeling at www.fmcna.com.

Other indications for use: stay•safe catheter, extension sets, stay•safe drain set

