

LIBERTY[®] Select Patient Skills Checklist

for software versions 3.0.2 and higher

Instructions

Use this form to track and document patient and care partner training on Liberty Select cyclor software version 3.0.2.

This checklist is designed to be used as a resource in conjunction with the Liberty Select cyclor User's Guide. Use a check mark to show steps performed during review and practice. Initials indicate all steps have been completed for each skill in that section. If the skill is not adequately demonstrated for a sub-topic, a follow-up plan of action is recommended. Do not leave fields blank. Enter N/A (not applicable) where appropriate.

CENTER NAME

Signatures

PATIENT NAME

PRINTED NAME	<input type="text"/>	INITIALS	<input type="text"/>
SIGNATURE	<input type="text"/>		

CARE PARTNER NAME

PRINTED NAME	<input type="text"/>	INITIALS	<input type="text"/>
SIGNATURE	<input type="text"/>		

TRAINING START DATE	<input type="text"/>	TRAINING END DATE	<input type="text"/>
---------------------	----------------------	-------------------	----------------------

► Setting up the LIBERTY Select Cyclor for Treatment (Tx)

SKILL	REVIEWED	PATIENT DEMONSTRATES SKILL	CARE PARTNER DEMONSTRATES SKILL (if applicable)
Demonstrates how to insert the IQdrive™ (USB) into the cyclor.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Locates white power switch on back of machine and sets it to ON position.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Checks treatment settings on the “Ready” screen.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Confirms treatment settings are correct and presses OK to start treatment.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Follows the step-by-step on-screen instructions exactly. Presses Next when indicated on each screen.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Gathers correct supplies for the treatment per center’s instructions.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Puts on mask and ensures that everyone else in the room puts on a mask.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Cleans hands thoroughly per center’s protocol.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS	<input type="text"/>	PATIENT INITIALS	<input type="text"/>	CARE PARTNER INITIALS	<input type="text"/>	DATE	<input type="text"/>
------------------	----------------------	------------------	----------------------	-----------------------	----------------------	------	----------------------

► Setting up the LIBERTY Select Cyclor for Treatment (Tx) (cont'd)

SKILL	REVIEWED	PATIENT DEMONSTRATES SKILL	CARE PARTNER DEMONSTRATES SKILL (if applicable)
Opens the cyclor set, checks expiration date, removes cassette and tubing using proper technique to avoid damaging cassette film.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Verifies that Liberty cyclor set is free of kinks and damage.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Locates and opens cassette door.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Using proper technique, inserts cassette up and under guide pins then snaps bottom of cassette into cassette catch. Closes the door.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Choose and confirm correct heater bag volume.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Places the patient line in the stay•safe® Organizer.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Connects the lines to solution bags when prompted to by the cyclor.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS		PATIENT INITIALS		CARE PARTNER INITIALS		DATE	
------------------	--	------------------	--	-----------------------	--	------	--

► Setting up the LIBERTY Select Cyclor for Treatment (Tx) (cont'd)

SKILL	REVIEWED	PATIENT DEMONSTRATES SKILL	CARE PARTNER DEMONSTRATES SKILL (if applicable)
Places largest volume solution bag on the heater tray in correct position. Follows screen instructions.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Identifies BACK button to return to heater bag volume selection screen.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Verifies the bags are connected per screen indicator.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Verifies that only one solution bag is on heater tray when moving to NEXT step.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Ensures blue clamp of patient line is open after flush is complete.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Ensures the patient line is fully primed. Prime a second time if integrated set is used or MTS are used.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Closes patient line (blue clamp).	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Ensure patient understands how to properly enter daytime exchange information, if applicable.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS	<input type="text"/>	PATIENT INITIALS	<input type="text"/>	CARE PARTNER INITIALS	<input type="text"/>	DATE	<input type="text"/>
------------------	----------------------	------------------	----------------------	-----------------------	----------------------	------	----------------------

► Setting up the LIBERTY Select Cyclor for Treatment (Tx) (cont'd)

SKILL	REVIEWED	PATIENT DEMONSTRATES SKILL	CARE PARTNER DEMONSTRATES SKILL (if applicable)
Connects the catheter extension set to the PIN connector. Opens patient line (blue clamp) and catheter extension set (white clamp). Removes PIN connector from the stay-safe Organizer.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Presses NEXT to start treatment.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

► Ending your Treatment

SKILL	REVIEWED	PATIENT DEMONSTRATES SKILL	CARE PARTNER DEMONSTRATES SKILL (if applicable)
Follows the step-by-step on-screen instructions exactly. Presses NEXT when indicated on each screen.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Turns the blue button on the PIN connector clockwise to unlock, then pushes the blue button to insert the PIN into the catheter extension set. Closes blue clamp on patient line and white clamp on catheter extension set.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS	<input type="text"/>	PATIENT INITIALS	<input type="text"/>	CARE PARTNER INITIALS	<input type="text"/>	DATE	<input type="text"/>
------------------	----------------------	------------------	----------------------	-----------------------	----------------------	------	----------------------

► Ending your Treatment (cont'd)

SKILL	REVIEWED	PATIENT DEMONSTRATES SKILL	CARE PARTNER DEMONSTRATES SKILL (if applicable)
Places a new stay•safe cap in the left side of the stay•safe Organizer.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Inserts the PIN connector that is connected to catheter extension set into the stay•safe clip of the Organizer.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Clean hands per center's protocol.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Using aseptic technique, remove the protective cover from the new stay•safe cap and sets it aside.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Unscrews the catheter extension set from the PIN connector.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Connects the catheter extension set to the new stay•safe cap.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Removes the catheter extension set from the Organizer notch. Makes sure the cap is tight.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS	<input type="text"/>	PATIENT INITIALS	<input type="text"/>	CARE PARTNER INITIALS	<input type="text"/>	DATE	<input type="text"/>
------------------	----------------------	------------------	----------------------	-----------------------	----------------------	------	----------------------

► Ending your Treatment (cont'd)

SKILL	REVIEWED	PATIENT DEMONSTRATES SKILL	CARE PARTNER DEMONSTRATES SKILL (if applicable)
Presses the NEXT button on the cyclor.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
When prompted on the screen, closes all clamps on the cyclor set. Removes the cassette from the cyclor and closes the door.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Presses NEXT to continue.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Reviews and explains each page of the treatment summary to the nurse.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Presses NEXT to continue.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Turns off the cyclor.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Cleans the cyclor with (1:100) bleach solution or other suitable hospital disinfectant. Rinse off with a water-dampened cloth.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS	<input type="text"/>	PATIENT INITIALS	<input type="text"/>	CARE PARTNER INITIALS	<input type="text"/>	DATE	<input type="text"/>
------------------	----------------------	------------------	----------------------	-----------------------	----------------------	------	----------------------

► General Machine and Disposables Knowledge

SKILL	REVIEWED		PATIENT DEMONSTRATES SKILL		CARE PARTNER DEMONSTRATES SKILL (if applicable)	
Identifies Heater/Heater Tray, Bag Sensor, IQdrive Slot/USB Port, Kinexus Gateway/Modem Connection Port, and Fan Vent, Control Panel, Door, and On/Off switch.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Demonstrates how to handle alarms by pressing STOP to mute, identifying alarm code, and troubleshooting as taught.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Can name the four hard key buttons.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
States the colors of the status bar and what each color indicates.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Demonstrates how to check "Treatment Settings."	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Demonstrates how to check "My Records."	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Demonstrates how to set "Options" for cyclor operation, including SI units.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>
Identifies the color of the cyclor set solution bag line clamps and what each color indicates.	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>	Yes <input type="checkbox"/>	No <input type="checkbox"/>

TRAINER INITIALS		PATIENT INITIALS		CARE PARTNER INITIALS		DATE	
------------------	--	------------------	--	-----------------------	--	------	--

► General Machine and Disposables Knowledge (cont'd)

SKILL	REVIEWED	PATIENT DEMONSTRATES SKILL	CARE PARTNER DEMONSTRATES SKILL (if applicable)
Identifies PIN connector on cyclor set and can point to the blue button and PIN .	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
If able, lifts the cyclor properly using the handles when moving it.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Demonstrates how to accept or reject the settings from the Kinexus Gateway (if applicable).	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Demonstrates how to accept or reject the settings from the IQdrive.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Demonstrates how to review current therapy settings, by using the Current Settings button.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Demonstrates how to change treatment parameters on the "My Settings" screen.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Presses OK to save changed settings prior to leaving a screen when a change has been made during programming.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Differentiates between connected and unconnected lines in the "Verify Bags" screen.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS	<input type="text"/>	PATIENT INITIALS	<input type="text"/>	CARE PARTNER INITIALS	<input type="text"/>	DATE	<input type="text"/>
------------------	----------------------	------------------	----------------------	-----------------------	----------------------	------	----------------------

► General Machine and Disposables Knowledge (cont'd)

SKILL	REVIEWED	PATIENT DEMONSTRATES SKILL	CARE PARTNER DEMONSTRATES SKILL (if applicable)
Troubleshoots alarms as instructed.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Identifies location of Technical Support number, software version, and serial number on cyclor.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Writes down the alarm code prior to calling Technical Support.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Describes the function of the Details button.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Progresses appropriately after a power failure.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS		PATIENT INITIALS		CARE PARTNER INITIALS		DATE	
------------------	--	------------------	--	-----------------------	--	------	--

► stay•safe connection and disconnection procedure for APD (P/N 100291-01)

SKILL	REVIEWED	PATIENT DEMONSTRATES SKILL	CARE PARTNER DEMONSTRATES SKILL (if applicable)
Identifies stay•safe PIN connector.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Demonstrates proper procedure when connecting to cyclor set.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Demonstrates proper procedure when disconnecting from cyclor set.	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>
Demonstrates proper placement of first and second PIN connector when using cyclor set with dual PIN connector or optional Multiple Tubing Segment (MTS).	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>	Yes <input type="checkbox"/> No <input type="checkbox"/>

TRAINER INITIALS		PATIENT INITIALS		CARE PARTNER INITIALS		DATE	
------------------	--	------------------	--	-----------------------	--	------	--

Indications for Use

The Fresenius Liberty Select Cyclor is indicated for acute and chronic peritoneal dialysis. The stay•safe PIN connectors are indicated for use in acute and chronic peritoneal dialysis.

Caution: Federal (U.S.) law restricts these devices to sale by or on order of a physician.

Use

The Kinexus Gateway is intended for transmission of patient data collected during peritoneal dialysis therapy with the Liberty Select cyclor.

Note: Read the Instructions for Use for safe and proper use of these devices. For a complete description of hazards, contraindications, side effects, and precautions, see full package labeling at [fmcna.com](https://www.fmcna.com).

Fresenius Medical Care North America
920 Winter Street | Waltham, MA 02451 | [fmcna.com](https://www.fmcna.com)

© 2022 Fresenius Medical Care. All Rights Reserved. Fresenius Medical Care, the triangle logo, Liberty, Kinexus, Liberty Select, IQdrive, and stay•safe are trademarks of Fresenius Medical Care Holdings, Inc. or its affiliated companies. All other trademarks are the property of their respective owners. P/N 103419-01 Rev C 07/2022



**FRESENIUS
MEDICAL CARE**