



# kinexus<sup>®</sup> Cellular Gateway

## Quick Start Guide

# kinexus Cellular Gateway Quick Start Guide

The kinexus Cellular Gateway is used to transfer treatment data to your dialysis center. The data transfer only occurs when the Liberty Select Cyclor is connected to the Gateway and a cellular connection is established.

**This guide is intended to provide patients with quick guidance to set up and connect the kinexus Cellular Gateway to the Liberty Select Cyclor.**

Please refer to the User Guide for warnings and more detailed instructions. This supplemental material does not replace the kinexus Cellular Gateway User's Guide. Users should review and refer to the User's Guide for complete information on this product.

**Have questions or issues? Contact Technical Support 800-227-2572**



**NOTE: Do not connect or disconnect the Gateway from Cyclor while the Cyclor is powered ON**

## Setting up the Gateway

1. **Open** package and **check** for missing or damaged parts.



2. **Tighten** the antennas on at the back of the Gateway until finger tight



- 3. Connect** the power cord to the Gateway and **plug** the power adapter into a wall outlet.

**Do not use a power strip**




- 4. Wait** approximately **3 minutes** for the **Status** LED to turn **green**. You may hear an audible signal “Gateway is ON”



- 5. Observe** the Cell Strength lights while slowly rotating the antennas on the back of the Gateway. Position them for the strongest cell strength



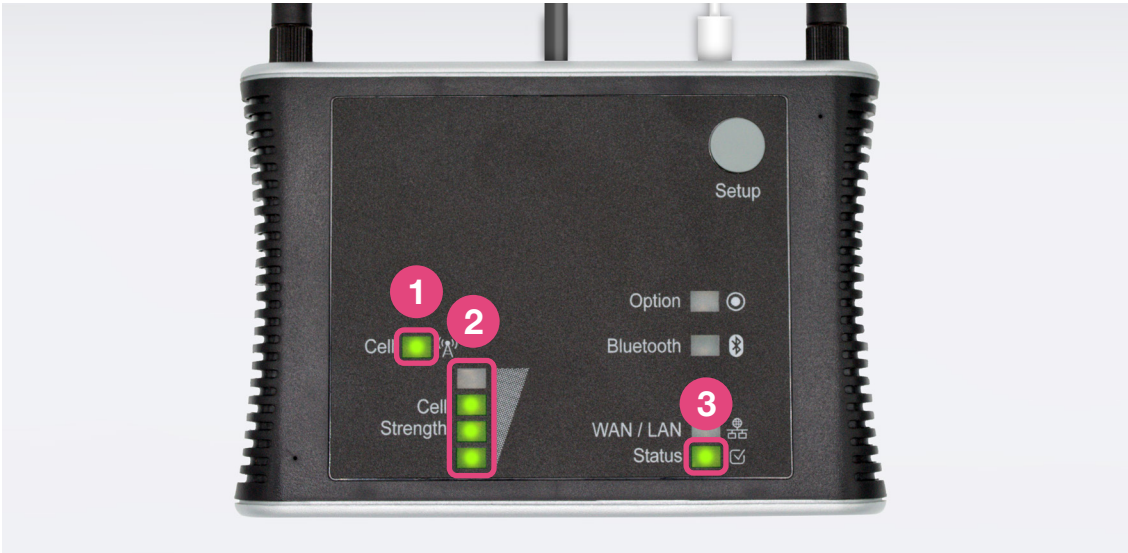

### 1 Cell LED

- On Connected to a cellular network
- Off Not connected to a cellular network

### 2 Cell Strength LED

- Best reception
  - 
  - 
  - 
  -
- Poor reception (try relocating device near a window)

No reception

### 3 Status LED

- Solid green Unit operates correctly
- Blinking green Data being received / transmitted
- Solid red System error occurred; reboot the device
- Red then green Unit is booting
- Off No power or unit is inoperative



# Connecting Liberty Select Cyclor to the Gateway

1. Ensure the Cyclor is powered OFF



2. Connect the RS-232 end of the cable to the Cyclor. Press between the thumb screws to firmly seat the connector



3. Turn both thumb screws clockwise until finger tight

Caution: **Do not use a screwdriver** to tighten the cable connection. Damage to your device may occur




4. **Plug** the USB end of the cable into the USB port on the Gateway

Note: The Cellular Gateway should be powered ON for **5 minutes** before turning on the Cyclor



5. **Turn on** the Liberty Select Cyclor

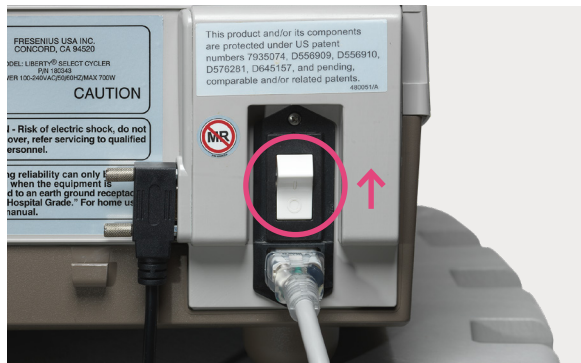



Read the Instructions for Use for safe and proper use of this device. For a complete description of hazards, contraindications, side effects, and precautions, see full package labeling at [www.fmcna.com](http://www.fmcna.com)

**Indications for Use:** The Fresenius Liberty Select Cyclor is indicated for acute and chronic peritoneal dialysis.

**Caution:** Federal (US) law restricts this device to sale by or on the order of a physician.

**Use:** The kinexus Gateway is intended for transmission of home dialysis therapy data and patient data for patients using certain Fresenius Medical Care products, including peritoneal dialysis cyclor(s).

kinexus is not an electronic health record (EHR) or billing solution.

© 2024 Fresenius Medical Care. All Rights Reserved. Fresenius Medical Care, the triangle logo, Liberty, and kinexus are trademarks of Fresenius Medical Care Holdings, Inc. or its affiliated companies. All other trademarks are the property of their respective owners.  
P/N 104271-01 Rev B 10/2024